Balanze

DA: Brugsanvisning	2
EN: Instructions for use	12
SV: Bruksanvisning	22
FI: Käyttöohjeet	32
ET: Kasutusjuhend	42
NL: Gebruiksaanwijzing	52

BALANZE
by ZiboCare

BALANZE

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RELAX AND SLEEP WELL WITH THE BALANZE DUVET AND PLAID

The Balanze duvet and plaid are newly developed quality products from ZiboCare. They have been developed based on our many years of experience in the healthcare sector and sensory stimulating aids, and in close collaboration with both users and healthcare professionals.

Always read all the instructions before use

IMPORTANT SAFETY INSTRUCTIONS AND CONTRAINDICATIONS

Please read all the instructions before use.

The weight duvet or blanket may only be used for its intended purpose, as described in these instructions.

Mentally disabled people and users who cannot take care of themselves or communicate their experiences with the Balanze duvet and blanket may only use the Balanze duvet and blanket under supervision.

The weight duvet or blanket is not suitable for adults or children who are not able to remove the weight duvet without difficulty.

In case of decreased lung capacity or heart problems: Consult relevant healthcare personnel before using the Balanze duvet or blanket.

In the case of surgical implants, osteoporosis: Consult relevant healthcare personnel before using the Balanze duvet or blanket.

The Balanze weight duvet or blanket must not be used near fire sources or hot surfaces such as a fire, burning cigarettes or hot lamps. Smoking is not recommended when using the Balanze weight duvet or blanket.

When using hygiene covers, be aware of the risk of suffocation.

The Balanze weight duvet or blanket can pose a trip hazard if it is lying on the floor

The Balanze weight duvet or blanket is unsuitable for children under 3 years.

HOW THE WEIGHT DUVET OR BLANKET WORKS

Purpose and use

The weight duvet or blanket are sensory stimulating aids that can be used to stimulate sensory and tactile nerve impulses.

Chains are sewn into the duvet in channels. This allows the duvet to exert an even, uniform and deep pressure on the user's body. The edges of the duvet are soft so it feels comfortable, without any sources of irritation.

The weight duvet or blanket wraps tightly around the body and gives a feeling of security. The sensory stimuli of the chains contributes to a better feeling of one's own body, and the relaxed feeling of security helps to create calm, balance and a better quality of sleep.

Examples of how to use the weight duvet or blanket:

WHO DOES THE WEIGHT DUVET OR BLANKET HELP?

Balance is a weight and tactile duvet or blanket, intended to help with many different needs and challenges, for people of all ages (*please read the important safety instructions and contraindications*).

It is especially good for people with:

- Sleep disorders, poor sleep quality or difficulty falling asleep
- Sensory integration problems
- Dementia with, among other things, restlessness, worry, anxiety and reversed circadian rhythm
- Developmental disorders (ADD, ADHD, Autism Spectrum Disorders, Tourette's, Asperger's, etc.)
- Mental imbalance and stress symptoms
- Concentration problems
- Mood swings

- Psychiatric disorders (anxiety, depression, schizophrenia, eating disorders, etc.)
- Neurological disorders (brain damage, Parkinson's, spasticity, etc.)
- Mental and motor restlessness including restless legs
- Racing thoughts, both during the day and at night

Balance duvet and blanket models:

Model	Size and weight	Item no.	HMI no.
Adult duvet, Classic	140x200cm, 4 kg	700032003	128898
Adult duvet, Classic	140x200cm, 6 kg	700032004	128901
Adult duvet, Classic	140x200cm, 8 kg	700032005	128904
Adult duvet, Classic	140x200cm, 10 kg	700032006	128907
Adult duvet, Classic	140x200cm, 12 kg	700032007	128910
Adult duvet, Classic	140x200cm, 14 kg	700032008	128913
Adult duvet, Sensitive	140x200cm, 4 kg	700032101	128916
Adult duvet, Sensitive	140x200cm, 6 kg	700032102	128919
Adult duvet, Sensitive	140x200cm, 8 kg	700032103	128922
Adult duvet, Sensitive	140x200cm, 10 kg	700032104	128925
Adult duvet, Sensitive	140x200cm, 12 kg	700032105	128928
Adult duvet, Sensitive	140x200cm, 14 kg	700032106	128931
Adult duvet, Classic, long	140x220cm, 6 kg	700032201	128935
Adult duvet, Classic, long	140x220cm, 8 kg	700032202	128938
Adult duvet, Classic, long	140x220cm, 10 kg	700032203	128941
Adult duvet, Classic, long	140x220cm, 12 kg	700032204	128944
Adult duvet, Classic, long	140x220cm, 14 kg	700032205	128947
Adult duvet, Sensitive, long	140x220cm, 6 kg	700032301	128950
Adult duvet, Sensitive, long	140x220cm, 8 kg	700032302	128953
Adult duvet, Sensitive, long	140x220cm, 10 kg	700032303	128956
Adult duvet, Sensitive, long	140x220cm, 12 kg	700032304	128962
Adult duvet, Sensitive, long	140x220cm, 14 kg	700032305	128959
Youth duvet, Classic	100x140cm, 3 kg	700032001	130465
Youth duvet, Classic	100x140cm, 5 kg	700032002	130468
Youth duvet, Sensitive	100x140cm, 3 kg	700032107	130471
Youth duvet, Sensitive	100x140cm, 5 kg	700032108	130474
Blanket	70x125cm, 4 kg	700032401	130477
Blanket	70x200cm, 5 kg	700032402	130480
Blanket	70x200cm, 7 kg	700032403	130483

The right model for you depends on your body type and the issues you are experiencing. For more information, contact our customer service staff.

HOW TO USE THE WEIGHT DUVET OR BLANKET

The weight duvet or blanket is used with a normal duvet cover and like a regular duvet. The cover protects the duvet and prolongs its service life.

Special hygienic and cotton duvet covers for Balance duvets are also available, and both feature three-way zips that make it easy to put them on a duvet; see accessories for more.

We also recommend using a cover with the Balance blanket. For those experiencing incontinence, and for use in environments with dust or dirt, we recommend using the hygienic cover; see accessories for more.

Use the weight duvet or blanket instead of a regular duvet. Most people feel comfortable sleeping with the weight duvet or blanket all night from the first night. Some users prefer to gradually work their way up to using the duvet all night. It may be a good idea to sit with the duvet over your legs for a while before going to bed, so you get used to it before going to bed, at least the first time.

Most people can use the Balance blanket right away with no adjustment period. However, some may find it more comfortable to start by placing the blanket on the legs and lap, then gradually moving the blanket up towards the chest and shoulders.

Please note that the weight duvet or blanket has two different sides. One side is soft and padded.

The other side has no padding, giving stronger stimuli from the chains.

Some people do not experience much difference in the effect between the two different sides, while for others there can be a significant difference depending on which side you use. It is recommended to experiment and find out what works best for you. As everyone's sensory systems are different, what works well for one user may not work well for another.

If you want stronger stimuli from the weight duvet or blanket, you can tuck/wrap it snugly around your body: possibly get somebody to help with that.

If you want the duvet or blanket to be cool before use, it must be placed in a cool place during the day. Conversely, keep it in a warm place if you want the duvet or blanket to be warm when it is to be used, which can be advantageous, for example, for patients with pain, very thin patients or people who are cold sensitive.

For those experiencing incontinence, we recommend placing the duvet or blanket in a hygienic cover to protect it and extend its lifetime. Using a cover will reduce the need to wash the blanket.

For those desiring a duvet softer than the Balanze Classic, we recommend the Sensitive model. This model features twice as much padding on the soft side as the Classic model. The extra padding may be beneficial for patients with pain, very thin patients, and those sensitive to the cold.

ACCESSORIES

The bag

The weight duvet or blanket comes in a nice, practical bag, which makes it easy to take the duvet with you everywhere - on weekends, at school camp, on holiday, etc. The bag also protects the duvet from dirt and moisture.

Cover with 3-way zip

Cover	Size	Type	Material
Cover for Balanze duvet, adult	140 x 200 cm	Standard	100% cotton
Cover for Balanze duvet, adult	140 x 200 cm	Hygienic	38% polyurethane / 62% polyester
Cover for Balanze duvet, youth	140 x 100 cm	Standard	100% cotton
Cover for Balanze duvet, youth	140 x 100 cm	Hygienic	38% polyurethane / 62% polyester
Cover for Balanze blanket	70 x 125 cm	Denim	100% polyester
Cover for Balanze blanket	70 x 200 cm	Denim	100% polyester
Cover for Balanze blanket	70 x 125 cm	Hygienic	38% polyurethane / 62% polyester
Cover for Balanze blanket	70 x 200 cm	Hygienic	38% polyurethane / 62% polyester

How to fold the duvet or blanket so that it fits easily in the bag :

CLEANING, MAINTENANCE AND WASHING

Washing instructions

The weight duvet or blanket is washed in an ordinary washing machine.
The need for and frequency of washing depend on its use and application..

- Always check before washing that The weight duvet or blanket is intact and that there is no damage or tears in the fabric or seams.
- If there is a tear in the fabric or seams, the duvet cannot be washed.
- The weight duvet or blanket is washed either in a laundry bag or in a duvet cover closed with a zip.
- Always remember to check the washing machine's instructions before washing, including weight limitation.
- If needed, the duvet can be split into two parts with the zip. Each half can be washed separately in a laundry bag or duvet cover.
- The weight duvet or blanket is washed on a gentle cycle at a maximum of 60°C and must be dried lying down.
- If the duvet is transferred to another user, it is recommended that the duvet is washed and the cover changed when the duvet is reused.

GENERAL INFORMATION

This equipment qualifies as medical equipment and bears a CE mark in accordance with EU Regulation 2017/745 of 5 April 2017.

The user and/or patient should report any serious incident that has occurred in connection with the device to the manufacturer and the competent authority of the Member State where the user and/or patient is located.

Storage

For maximum protection of The weight duvet or blanket, store it in the included bag, which protects it against moisture and dirt. The weight duvet or blanket must always be stored dry. Make sure that The weight duvet or blanket does not lie on the floor, where there is a risk of tripping over it.

Labelling

The weight duvet or blanket is labelled in accordance with EU legislation for medical devices class 1, including, among others, product name, production date, item number, size/weight and CE marking. The label with relevant information, including washing instructions, is sewn into the duvet so that the information is always easily accessible.

Warranty

Purchases are covered by a two-year warranty for manufacturing defects in accordance with the European Sales Law. The warranty does not cover misuse and ordinary wear. Lacking or improper product care, rough or improper use, and overloading of the duvet or blanket will void this warranty.

Service life

The product has an expected service life of 5 years based on correct use, handling, washing and storage as described in the instructions. The service life can be shortened by frequent washing. It is the user's responsibility to continuously assess whether the product can be used beyond the stated service life.

Service and maintenance

The weight duvet or blanket does not require any service or maintenance other than as stated for cleaning, maintenance and washing.

Recycling and disposal

- We think about the environment and work based on the UN's Global Goals. When disposing of the duvet, it can be divided into the following individual parts that can be recycled:
 - Iron

- Fabric
- Fibre filling

Waste is sorted and disposed of according to local rules.

Returns and complaints

In the event of a return or complaint, please contact the shop where the duvet was purchased. Here, they will help with the further handling of the case according to applicable terms and conditions.

Symbolforklaring

	Warning, see important safety instructions		Max. washing temperature 60°C. Gentle wash
	Manufacturer		Do not bleach
	Date of manufacture as stated on the duvet label		Do not iron
	Lot no.		Do not tumble dry
	Medical device		Chemically clean gently, using only petroleum
	See the instructions		Dry lying down
	CE-label - Class 1, MDR 2017/745		Sensory stimulated
	Reference number		

TECHNICAL SPECIFICATIONS

Dimensions:	See product options
Filling weight:	See product options
Filing:	90% coated steel chains/10% polyester fiber
Bolster:	100% cotton Oeko-Tex, Standard 100
Class:	Medical Device class 1 - 2017/745 af 5. april 2017

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ZiboCare A/S