
Sens-Aid

by ZiboCare®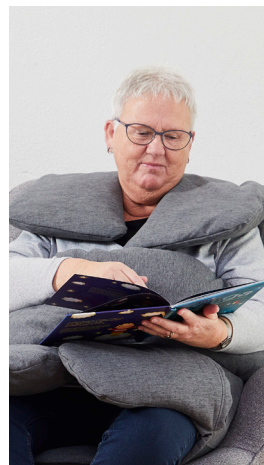

DA	Brugsanvisning.....	4
EN	Instructions for use.....	8
DE	Gebrauchsanweisung.....	14
SV	Bruksanvisning.....	21
FI	Käyttöohjeet.....	26
ET	Kasutusjuhend.....	32
NL	Gebruiksaanwijzing.....	38

DA - Brugsanvisning

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EN - Instructions for use

Important safety rules and contraindications

 Always read the instructions for use carefully before using the product.

WARNING!

Sens-Aid may only be used for its intended purpose and may not be combined with any accessories other than as described in this manual or in any other documentation provided by ZiboCare.

The product may not be used near or in connection with fire sources/hot surfaces, such as open fire, burning cigarettes or hot lamps. Smoking is not recommended when using Sens-Aid.

Do not use Sens-Aid when lying down. The enclosed hood should also not be used when lying down.

We do not recommend use in a standing/walking position, as Sens-Aid has been developed for use in a sitting position; it is not suitable for use other than on a chair, sofa, wheelchair, armchair or other types of seating.

Inspect Sens-Aid regularly to make sure it is not damaged. If Sens-Aid is damaged, do not use it until the damage has been repaired.

Sens-Aid should only be used under supervision for the mentally handicapped or users who are unable to communicate their experiences with the product.

Sens-Aid and the hood is not suitable for adults or children who are not able to remove it without difficulty.

When using covers and the hood, be aware of the risk of suffocation.

In case of surgical implants, osteoporosis, reduced lung capacity, heart problems or underweight users, e.g., in case of eating disorders: Consult your own doctor before using Sens-Aid.

Sens-Aid and the hood should not be used for children under 3 years old.

Be careful not to trip over Sens-Aid or the hood if it is lying on the floor.

1. Introduction

1.1 General information

This product is a medical device and CE marked according to EU Medical Device Regulation MDR 2017/745.

According to the regulations of the authorities regarding medical devices, it is the duty of the manufacturer and healthcare professionals to report any accidents and serious adverse events involving their products. We kindly ask that all information on any accidents and serious adverse events involving our products be reported immediately to ZiboCare and the competent authority of the Member State where the user and/or patient is established.

1.2 Intended use

Sens-Aid is a weighted blanket intended to be used at home, at school or in institutions when there is a need for the stimulation that Sens-Aid provides, primarily in order to create a sense of calm, physically and mentally.

Sens-Aid has six heavy wings that wrap around the body, according to the user's individual needs and preferences.

By using Sens-Aid, you may experience a reduction in restlessness, racing thoughts, anxiety and insecurity, including an overall feeling of balance and well-being. The weight of the wings hugs, gathers and calms the body. This is because the hormone oxytocin is released and sends signals to the brain that it is safe to "gear down", calm down and recharge.

Sens-Aid is used in a chair, for example, an armchair, school chair, wheelchair, during transport or in a sofa – in both indoor and outdoor areas. It is recommended to use a chair with a backrest.

Hood:

Can be purchased as an accessory.

Use on the head with the face free.

The hood can reduce auditory and visual stimuli from the surroundings.

1.3 Typical applications

Sens-Aid is intended for children and adults of all ages experiencing a general failure to thrive, burnout and a reduced amount of energy. The adult model is recommended for teenagers and adults and the junior model is for children aged 4 and above. The user may have a diagnosis, but Sens-Aid also caters to people with diverse challenges, even without a diagnosis. Sens-Aid is often used for:

- Motor restlessness
- Mental imbalance
- Eating disorders
- Concentration issues
- Neurological disorders
- Dementia
- Brain Injury
- Mental disorders, e.g., restlessness and anxiety
- Neuropsychiatric diagnoses, e.g., ADHD, ADD, Asperger's syndrome, autism and Tourette syndrome
- Spasticity
- "Racing thoughts"
- Involuntary movements
- Social anxiety

1.4 Sens-Aid variants

When folded, Sens-Aid does not take up much space and is easy to carry in the transport bag.

Sens-Aid is available in the following versions:

Size and weight	REF	Cover colour
Adult, 7 kg (incl. cover, bag)	SA800008	Heather grey
Junior, 5 kg (incl. cover, bag)	SA800012	Heather grey
Hood - Adult	8098774	Heather grey
Hood - Junior	8098775	Heather grey

Junior is for kids.
Adult is for grown children/teenagers and adults.

2. Features and positions

Once you have sat down in the chair on top of Sens-Aid, you can place the wings over the body to suit your individual preferences and needs. You can either do it yourself or have someone else assist you.

This is how it is done:

1
Placement: Place the folded Sens-Aid in the chair, ensuring that the back with the anti-slip surface faces down before sitting down. Spread the wings over the chair. The upper wings may be folded upward over the back of the chair. The middle and lower wings are folded over the sides of the chair and possibly the armrests.

2
Sit down in the chair on top of Sens-Aid.

3
The lower wings are placed on the thighs. The pockets can be used if you need to calm your arms and hands.

4
The middle wings are placed around the stomach. In this case, you may also use the pockets for your hands and arms and feel the calm that arises when you "gather your body" with your arms in the pockets.

5
The upper wings are placed on the shoulders so that the weight is experienced on the shoulders and the chest. This helps to deepen breathing and has a calming effect on both the body and the head.

6
The hood (accessory) is placed on top of the head with the face free. The hood can reduce auditory and visual stimuli from the environment that can distract and create anxiety.

ACCESSORIES

Sens-Aid comes in a practical and smart bag with a shoulder strap, which makes it easy to take Sens-Aid with you anywhere you need it. At the same time, the bag protects Sens-Aid from dirt and grime. A washable cover is included with the purchase of Sens-Aid or the hood.

For many, it can be both the most practical and hygienic option to have several covers for Sens-Aid and the hood, e.g., if one Sens-Aid is being used by several users, or to ensure there is always a clean and dry cover available. If the cover is damaged or if extra covers are needed, they can always be purchased separately, either on the ZiboCare website or by contacting customer service.

How to easily fold Sens-Aid to fit it into the included bag:

3. How to unpack Sens-Aid

Sens-Aid is delivered wrapped in recyclable plastic.

1. Start by checking that the packaging is undamaged.
2. Remove the plastic bag and any protective material. Avoid using a knife when opening the packaging as this may damage Sens-Aid and the included bag.
3. It might be a good idea to unfold Sens-Aid in a chair, see the instructions in Section 2
4. Be sure to save these instructions for use, as well as the cleaning information for future reference.

4. Cleaning

It is important to follow the instructions below when washing/cleaning the product, e.g., before using Sens-Aid for a new user. If in doubt, contact your healthcare provider in charge of hygiene-related matters or your supplier for assistance and further instructions.

The Sens-Aid core and hood are made from PU material, which may be disinfected or wiped off, but cannot be washed in a washing machine.

The Sens-Aid PU core and hood may be wiped off using a mild cleaning solution or disinfected using a suitable solution, e.g., alcohol with or without surfactants, isopropanol 70%, an oxidizing solution or max. 1% chlorine. Stool and blood stains should be removed immediately with cold water. Carefully follow the local guidelines and the instructions for use of cleaning agents. Chlorine- and phenol-based cleaning agents may have a negative impact on the PU surface and should be avoided.

If chlorine is used, a mixture of maximum 1% is recommended.

The Sens-Aid textile cover and the cover of the hood can be washed in a normal washing machine on a gentle cycle at a maximum of 60 degrees Celsius.

PU core:

Cover:

Always check before cleaning that the Sens-Aid cover is intact and that there is no damage or breakage of fabric or seams.

5. Miscellaneous

Storage

Store Sens-Aid either in the transport bag, in a chair or folded in a place where there is no risk of tripping over it. Sens-Aid must always be stored indoors in a dry place.

Marking

Sens-Aid is marked in accordance with EU legislation for Class I medical devices, including product name, production date, size/weight, CE marking, LOT and REF numbers.

Warranty

Sens-Aid is covered by a 2-year warranty. The product is covered by a warranty for production defects. The warranty does not cover ordinary wear and tear, product deterioration or damage resulting from careless handling or care of the product.

Service life

The product has an expected service life of 5 years based on correct handling, maintenance and storage in the included cover.

Service and maintenance

Sens-Aid requires no service or maintenance other than as stated in these instructions for use.

Repair

If the surface of the Sens-Aid PU core is damaged or broken so that the filling comes out, the device may no longer be used. Contact the supplier for more information about possible repair.

Recycling and disposal

Upon disposal, Sens-Aid can be split into the following individual parts, which can be recycled:

- Plastic pellets
- Polyester
- Polyester with PU coating
- Polyurethane

Waste is sorted and disposed of according to local regulations.

Returns and claims

In the event of a return or claim, please contact the supplier. When returning a product, please follow the supplier's instructions. A return may only take place when the claim has been made. Damage that occurs during transport must be reported to the transport company.

Symbol legend

	Warning, see under important safety rules		May be wiped off/disinfected
	Manufacturer		Do not wash
	Date of production as indicated on Sens-Aid label		Do not bleach
	Lot no.		Do not iron
	Medical device		Do not tumble dry
	See instructions for use		Do no dry clean
	CE marked according to Class 1, MDR 2017/745		Do not heat in microwave or similar device
	Reference number		Wash on a gentle cycle at a maximum of 60 degrees Celsius
			Chlorine, <1%

TECHNICAL SPECIFICATIONS

Sens-Aid

Weight: Core with cover: Adult - 7 kg, Junior - 5 kg
 Cover: 100% polyester Oeko-Tex, Standard 100
 Filling: 100% plastic pellets
 Core: 100% polyurethane
 Patented: WO 019/219152

Hood

Cover: 100% polyester
 Filling: 100% polyester
 Core: 100% polyurethane

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